


Testing for home care workers

How testing works

Complete a swab test on the same day each week between Thursday and Sunday. Make the state register, register and post the test on the same day. Make sure you have everything you need to fore, bu complete the test, including a Royal Mail return label.

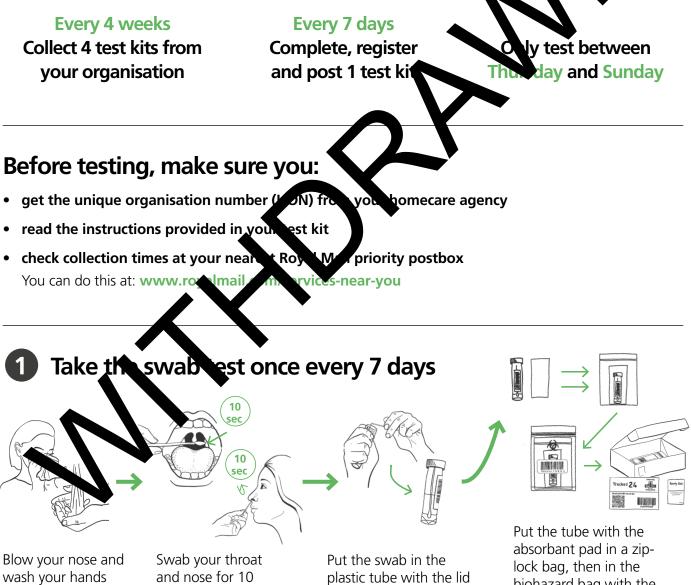

securely fastened. The

swab should fit inside

without bending.

lock bag, then in the biohazard bag with the matching barcode. Package the biohazard bag in the provided return box, secure with the security seal, and stick on the Royal Mail return label.

before testing.

seconds each.

Register your test kit online

Register your test kit online at: https://organisations.testfor-coronavirus.service.gov.uk/register-organisation-tests

To register you will need your homecare agency's unique organisation number (UON)



3

Post your test kit by Royal Mail priority postbox on the same day of testing

PRIORITY PC

шип

Post the completed kits no later than **1 hour** before the last collection time (weekend times may differ from weekdays). **Do not** complete or post your test on a bank holiday

Only use a Royal Mail priority postbox

with the label and NHS logos below:



NHS

Some priority postboxes have a Sunday collection. They'll show one of these labels:

After testing

3-5 day. Conduct 1 test every 7 days. You should receive your re

psitive regult you must lf you q ťα

a face ma immediately is you're outside your home

r manager rm 🛛

> pme/immediately return home and self isolate for 10 days from You may return to work on day 11 if you've not had a high temperature for 48 hours and are well.

If you have any questions you can

- speak to your manager ٠
- call **119**

Do not delay getting help if you're worried. Trust your instincts.

For medical help, contact 111. In an emergency dial 999. For more advice go to www.gov.uk/coronavirus