

Appendix


Below is part of the instructions for use provided with these models by Draeger Medical, which explains how to correctly fit both sets of side panels.

Fitting the panels

- 1 Insert the lower mounting pins of the panel hinge-pieces into the guide slots on either side.

With the panels on the long side, ensure that the handle points outwards.

- Position the panel semi-upright,
- 2 press the panel down firmly until the pins lock into position at the bottom of the slots.
- Fold the panel up into the vertical position and allow it to drop into the locking position.
- Fit all four panels in this way.

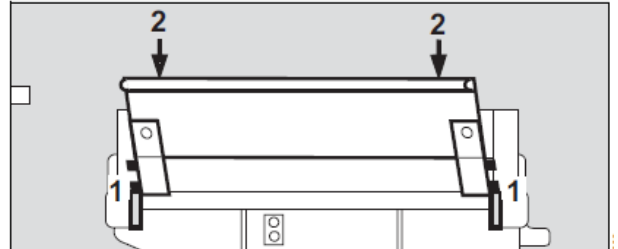

Make sure that the panels are seated correctly.

To open:

- Lift the panel all the way up until the upper pins come out of their slots, then fold the panel down.

With lively infants or older children:

- Use 230 mm high panels.

Fitting the inner panels

WARNING

**Inner panels should be used with all patients, regardless of whether the side panels are up or down!
Otherwise the patient may fall out!**

- Insert the pins into the tapered holes in the corners of the housing and press down lightly until the inner panel is firmly in place.

Make sure the panels are seated correctly.

