


Appendix

 Edwards	Edwards Lifesciences ABP Business Unit Technical Service	Bulletin Nr. 134 Rev.A
--	---	------------------------

TOOLS REQUIRED

NW-H008-00

Expected time: 5 minutes

CAUTIONS

1. Only trained and qualified personnel should service this instrument. Refer to a copy of the Aquarius Service Manual as instructed.
2. Disconnect the power cord from the unit before removing the rear cover.
3. Disconnect the internal battery when working in the display or power distribution module areas to avoid potential damage to the instrument or exposure to high voltage.
4. Take the standard precautions to protect the instrument from ESD (electrostatic discharge) when handling any of the printed circuit board assemblies or programmed devices.
5. Retrieve all loose hardware from the unit.

CHECKING AND FIXING THE ROLLER

Make sure that the rollers are securely attached to the frame of the Aquarius. Use the NW-H008-00 tool to tighten the roller bolt to the frame. The roller needs to be securely attached to prevent tipping of the machine.



Image

1

Confidential information- not to be used or distributed without Edwards Lifesciences prior written approval