

How to use your rapid lateral flow test

Remember to **always read the instructions** that come with the test kit

	COVID-19 Self-Test (Rapid Antigen Test)	ACON Flowflex
<p>Swabbing process</p>	<p>4 times each tonsil, 10 circles, 1 nostril</p>	<p>3 circles, each nostril</p>
<p>Swab in tube</p>	15 secs	30 secs + rotate 5 times
<p>Resting time</p>	None	None
<p>Drops on device</p>	2	4
<p>Wait time</p>	<p>IMPORTANT</p> <p>Do not leave the test to develop for longer than 30 minutes as this will make the result void.</p>	<p>IMPORTANT</p> <p>Wait for 15 minutes to read the result. Do not read the result after 30 minutes.</p>

You've only completed your test once you've reported the result

Report your result

Scan the QR code or type in <https://www.gov.uk/report-covid19-result> to get to the reporting page

Report **all** results whether they are **positive**, **negative** or **void**

You will need:

- the code on the test cassette
- your organisation's unique organisation number



Your **Organisation Number (UON)** is